
MOIST BURN PADS

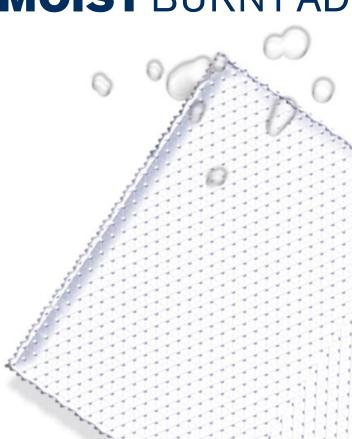

Institutional SKUs



Medical SKUs



Medical SKUs

Catalog Number 47-001 1 ½" x 2" (6 Pack)
Catalog Number 47-019 2" x 3" (4 Pack)
Catalog Number 47-027 3" x 4" (3 Pack)

Institutional SKUs

Catalog Number 47-122 1 ½" x 2" (25 Pack) Catalog Number 47-121 2" x 3" (25 Pack) Catalog Number 47-120 3" x 4" (25 Pack)



TARGET USER:

General users with compromised skin due to 1st or 2nd degree burns.

RECOMMENDED FOR:

Treatment of 1st or 2nd degree burns, cuts, scrapes, and bites.

COMPONENTS:

High-water content sterile hydrogel – made with 95% pure water.

FEATURES & BENEFITS:

- Won't stick to the wound
- · Helps prevent scarring
- · Soothing and cooling
- Non-irritant
- Odorless
- Provides instant relief
- Protects and cushions

MOIST BURN PADS

INSTRUCTIONS:

- 1. Clean the burn or wound area.
- 2. Remove and discard blue film.
- Gently apply the exposed moist pad over burn or wound.
- 4. Cover with adhesive tape or stretch gauze.
- 5. Change daily. If the pad becomes dry, moisten the pad then remove.
- 6. Do not use if the pouch is torn or damaged.

CAUTION: Do not use on third-degree burns. If complications occur, consult a physician. Do not use product if it is dried out.



MEDICAL SKUS

Catalog Number	Product Size	Product Per Package	Individual Package Size	Case Size	Quantity Per Case
47-001	1 1/2" x 2"	6	1 1/2" x 2 3/8" x 4 5/8"	7 1/2" x 6 1/4" x 4 3/4"	12
47-019	2" x 3"	4	1 1/2" x 2 3/4" x 5 5/8"	8 7/16" x 6 5/16" x 5 11/16"	12
47-027	3" x 4"	3	1 1/2" x 4" x 6 1/8"	9 1/2" x 8 3/4" x 6 1/2"	12

INSTITUTIONAL SKUS

Catalog Number	Product Size	Product Per Package	Individual Package Size	Case Size	Quantity Per Case
47-122	1 ½" x 2"	25	3" x 3 1/2" x 3 1/2"	10 3/4" x 7 3/16" x 6 7/16"	12
47-121	2" x 3"	25	3 1/4" x 4 1/2" x 4"	12 1/4" x 9 3/16" x 6 1/8"	12
47-120	3" x 4"	25	4 1/2" x 5 1/2" x 4 1/2"	13 3/4" x 11 3/16" x 9 3/8"	12

©2011 Spenco Medical Corporation. All Rights Reserved. ® Registered and ™ Trademark of Spenco Medical Corporation

